


T² Remote Pocket User Guide

You may use a cell phone, cordless telephone, or corded telephone to adjust your hearing system. The telephone MUST “beep” when the keys are pressed for this function to work.

- Hold the phone approximately 3 inches from the ear
- Press ** to “wake” T² (a tone series indicates that it is ready for a command)
- T² will “sleep” after 20 seconds
- Press the telephone key that corresponds to the desired function

Use the following keypad buttons to remotely adjust your hearing aid settings.

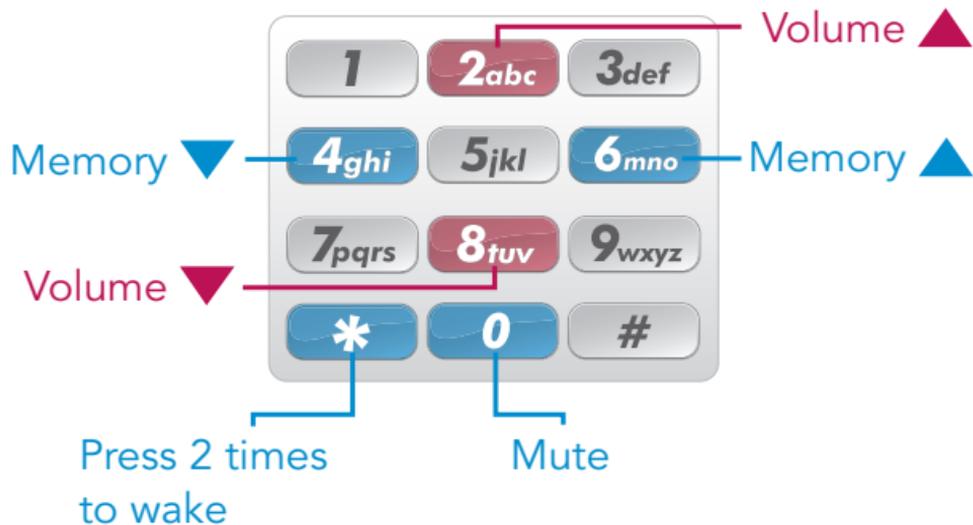

T² Remote Pocket User Guide

Troubleshooting Guide

SYMPTOM	POSSIBLE CAUSES	SOLUTIONS
Hearing system not responding to telephone command	Telephone keys not emitting beep tones	Consult telephone operator's manual to enable beep tones
	T ² not unlocked prior to command	Press ** and listen for unique T ² indicator
	Telephone too far from hearing system	Bring telephone closer to hearing system microphone
	Incorrect key command	Press ** to wake then press 2 or 8 to increase or decrease volume or 4 or 6 to change programs if device has multiple programs
Hearing system responded but won't respond to later command	T ² automatically "sleeps" 20 seconds after last command	Press ** to wake then press key that corresponds to desired action