

Inspire[®] X System Requirements


| GOVERNMENT SERVICES |



Prior To Install

For successful Inspire[®] X installation and operation, please make sure your computer has the following:

- **Processor Speed:** Pentium[®] 4 or higher recommended
- **System RAM:** 2 GB minimum, 3 GB recommended
- **Operating System:** Windows[®] 7, Windows 8 or 8.1, Windows 10
- **Browsers (for Online Ordering):**
 - Internet Explorer[®] - latest version
 - Chrome[™] - latest version
 - Firefox[®] - latest version
 - Microsoft Edge[®] - latest version
- **Communications Port for Programmer:**
 - Serial COM port (or an approved USB to Serial converter) is required for HiPro[®] and HiPro 2[®]
 - USB port for SpeedPort[™], SpeedPort 2, NOAHlink[®], HiPro USB, SurfLink[®] wireless programmer, TruLink[®] wireless programmer, 2.4 GHz Wireless Programmer, and NOAHlink wireless programmer
- **Audio Hardware:** Sound card and stereo speakers are recommended to take full advantage of media sounds
- **Client Database:** NOAH[®] System 3.x or 4, ProHear Deluxe or Standard, ProHear.NET, PatientBase[™]

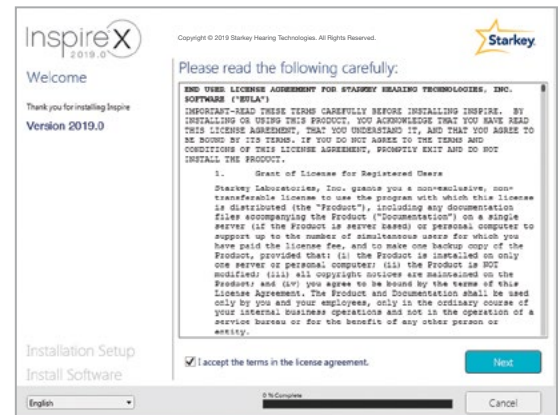

Installation Instructions

Inspire X installs directly into the supported client database and does not require install from within the database. The application installs from the USB drive, DVD or Inspire X Updater.

If prompted to restart the computer during this process, please continue to follow the prompts after restart until the "Finish" prompt appears. Insert the USB drive into the USB port or the Inspire X DVD into the DVD drive to begin the installation. If the installation does not begin automatically, browse the contents of the device and double-click the Setup file.