

HALO 2 INITIAL FITTING PROTOCOL

PRODUCT AVAILABILITY: Halo 2 Devices

Halo 2 programming must be completed prior to pairing the hearing devices to an Apple or Android device.

GET STARTED

1. Turn **Bluetooth OFF** on Apple or Android device.
2. Insert fresh **battery** into hearing devices and close battery doors.
3. Launch **Inspire X**.
4. On the **Get Started** screen, select **View Programmers**, then choose the **TruLink programmer** or the appropriate non-wireless programmer and select **OK**.
NOTE: Expected range for wireless programming is 30 feet.
5. Select the **check box** next to the hearing device(s) to be programmed.
6. Select **Read Device Settings** to begin a new fitting or for a follow-up fitting session using the current settings in the hearing devices, or **Load a Previous Session** to retrieve a previously saved session, then select **Start**.

Halo 2 features extended bandwidth, additional channels and bands, a dedicated music memory, advanced classification to manage all environments, and an on-board tinnitus stimulus.


Halo 2 devices connect directly to Apple and Android devices.

For the most up-to-date information on devices that are supported for use with TruLink, visit www.TruLinkHearing.com


INITIAL FIT AND AUTO PATH

Initial Fit launches automatically when programming new hearing devices.

1. Select the appropriate **Acoustic Options**, if applicable, then select **OK**.
2. Select the appropriate **Experience with Amplification**. Refer to the *Experience Manager QuickTIP* for additional information.
3. Insert hearing devices into the patient's ears, reduce all sources of noise, then select **Begin** to run the Auto Path.
NOTE: If hearing device programming is conducted prior to the fitting appointment, re-run the Feedback Canceller with the hearing devices in the patient's ears.
4. Review the Auto Path results screen. Select **Restart** if Status is Incomplete for any steps, or select **Close** to continue.

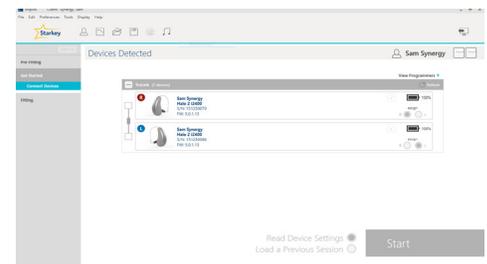

Get Started

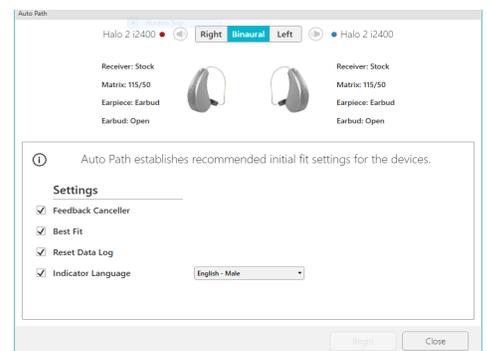

Auto Path

USER CONTROLS

The User Control Configuration demo box will appear upon initial programming with Inspire X. Select **Show Demo** to view a demonstration of user control configuration options, or select **Don't show again** and **Close**. Refer to the *User Controls QuickTIP* for additional information.

QUICKFIT AND FINE TUNING

- » Select **QuickFit** on the left navigation bar to make global fitting adjustments. Refer to the *QuickFit and Fine Tuning QuickTIP* for additional information.
- » Select **Fine Tuning** on the left navigation bar or from the bottom of the QuickFit screen to make detailed fitting adjustments of individual channels and input levels. Refer to the *QuickFit and Fine Tuning QuickTIP* for additional information.



Fine Tuning

