


MUSE INITIAL FITTING PROTOCOL

PRODUCT AVAILABILITY: Muse and SoundLens Synergy Wireless Devices

GET STARTED

1. Launch **Inspire X**.
2. On the **Get Started** screen, select **View Programmers**, then choose the appropriate programmer and select **OK**.
3. Connect hearing devices to the fitting computer via the appropriate programming accessories. Refer to the **Cables and Connectors Guide** for additional information regarding connecting the hearing devices. **NOTE:** All standard products are initially branded as left hearing devices. If you are fitting a hearing device on the right ear, select **Assign Right** to read as a right hearing device.
4. Select the **check box** next to the hearing device(s) to be programmed.
5. Select **Read Device Settings** to start a new fitting session using the current settings in the hearing devices or **Load a Previous Session** to retrieve a previously saved session, then select **Start**. **NOTE:** Select **Performance Update**, if applicable, to update the firmware of the devices.
6. Select the appropriate **Acoustic Options**, then select **OK**.

A product line featuring extended bandwidth, additional channels and bands, ear-to-ear phone streaming, a dedicated music memory, advanced classification to manage all environments, tinnitus stimulus in all devices, CROS/BiCROS devices, and wireless connectivity to SurfLink accessories.

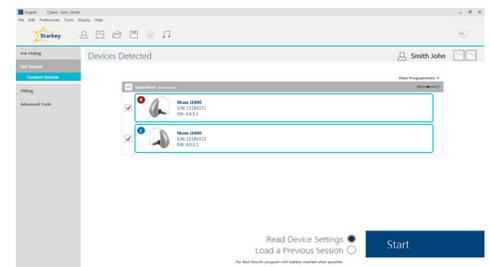


Get Started

AUTO PATH

Auto Path establishes recommended first fit settings for the connected devices based on the patient's audiogram. Functions include Feedback Canceller, Best Fit, Reset Data Log, Voice Language, and Experience Level.

1. Insert hearing devices into the patient's ears, reduce all sources of noise, then select **Begin**.
2. Select the desired **Experience Level**, then select **Next**.
3. Review the Auto Path results screen. Select **Restart** if Status is Incomplete for any steps, or select **Close** to continue.

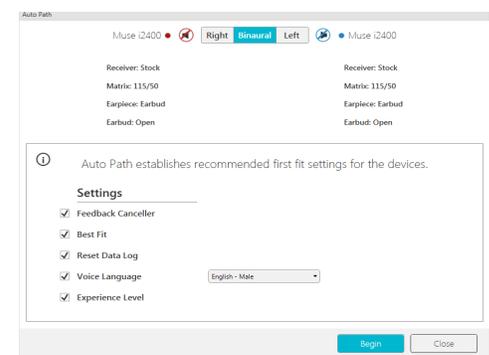

Auto Path

USER CONTROLS

The User Control Configuration demo box will appear upon first fitting with Inspire X. Select **Show Demo** for demonstration of user control configuration options, or select **Don't show again** and **Close**. Refer to the **User Controls QuickTIP** for additional information.

QUICKFIT AND FINE TUNING

- » Select **QuickFit** on the left navigation bar to make global fitting adjustments. Refer to the **QuickFit and Fine Tuning QuickTIP** for additional information.
- » Select **Fine Tuning** on the left navigation bar or from the QuickFit screen to make detailed fitting adjustments of individual channels and input levels. Refer to the **QuickFit and Fine Tuning QuickTIP** for additional information.



Fine Tuning

MEMORIES

1. Select **Memories** on left navigation bar or select the **Memories bar** in the middle of all fitting screens.
2. Select **drop-down arrows** to extend the Memories menu, then select the desired memories to program.
3. If configuring a **Music Memory**, refer to the *Music Memory QuickTIP*.
4. If enabling a **Telephone, Autophone, or Loop Memory**, select **Configure Phone, Loop, and DAI Options**. Refer to the *Ear to Ear Phone QuickTIP* for additional information.
NOTE: Select **microphone offset** if enabling a Loop or Phone memory.
5. Select **Additional Configuration Options** to Copy, Move, or Link memories and adjust the Telephone Release.




Memories

ACCESSORIES

Select **Accessories** on the left navigation bar to configure SurfLink accessories. Refer to the appropriate *SurfLink QuickTIPs* for additional information.

INDICATORS

1. Select **Indicators** on the left navigation bar.
2. Select or de-select **check boxes** to enable or disable indicators.
3. Select **Adjust All Indicators** to increase or decrease volume level for all Tone or Speech/Chimes indicators in 5 dB increments.
4. Select the **Individual Indicator names** to adjust the volume level and stimulus type for individual indicators, then select **Play** to play indicator directly through devices or **Demo** to play indicator through loudspeaker for demonstration purposes.

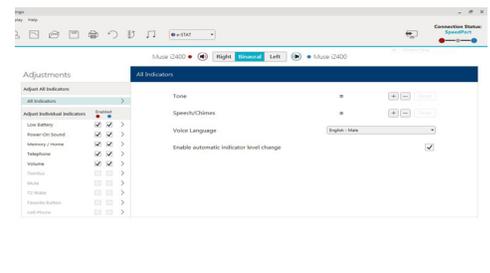


Indicators

ADDITIONAL ADJUSTMENTS

Refer to the *QuickTIP* library for additional information regarding programming of Experience Manager, Frequency Lowering, Environment Manager, and Tinnitus.

FITTING SUMMARY

1. Select **Fitting Summary** on the left navigation bar.
2. Select **feature names** or **details** to change status of features if necessary. Feature names are hyperlinks and will navigate to the adjustment screens.
3. Select **Print** button to generate a one-page printed report.
4. Select **Save** button to save current fitting session.

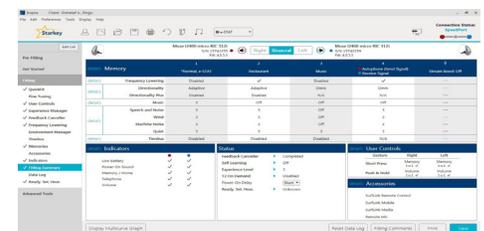

Fitting Summary

READY. SET. HEAR.

Ready. Set. Hear. is a one year triggered email program created to help new patients successfully transition into their new hearing devices. Contact your Marketing Representative for additional information.