

Starkey Hearing Technologies Remote Microphone +




Pairing

Starkey® Hearing Technologies Remote Microphone + must be paired to interact with Livio™ AI and Livio hearing aids. Pairing occurs once, then the hearing aids and Remote Microphone + will automatically recognize each other. The Remote Microphone + can be paired within Inspire® or outside of Inspire. Livio AI and Livio hearing aids can be paired with up to 5 SHT accessories, but only one can be actively streaming. The Remote Microphone + can pair with up to 8 Bluetooth® devices.*

Pairing within Inspire


- 1 Launch Inspire, connect and detect hearing aids.
- 2 Select “Accessories” on the left navigation bar.
- 3 Ensure the Remote Microphone + is powered on.
- 4 Place the hearing aids within 6 inches of the Remote Microphone +.
- 5 Place the Remote Microphone + in pairing mode by pressing the Input Mode and Volume Up buttons simultaneously until the LED status indicator alternately blinks blue and red.
- 6 Select “Pair” to put the hearing aids into pairing mode (A).
- 7 Pairing is successful when the Remote Microphone + image displays in the accessory screen (B).
- 8 Use the drop-down menu below the streaming accessories box to set “Hearing Aid Mic While Streaming”. Mic Reduced (50%) is the default (C).
- 9 Select the Information icon for pairing instructions if needed (D).

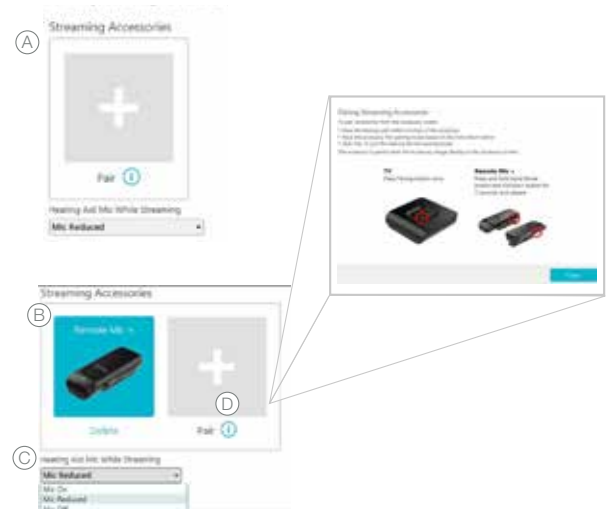

Pairing (Continued)


Pairing outside of Inspire

- 1 Power off any mobile phone or other accessories that were previously paired with the hearing aids.
- 2 Ensure the Remote Microphone + is powered on.
- 3 Open and close the battery doors of the hearing aids.
- 4 Place the hearing aids within 6 inches of the Remote Microphone +.
- 5 Press the Input Mode and Volume Up buttons simultaneously for 7 seconds until the LED status indicator alternately blinks blue and red.
- 6 A green LED will blink for each hearing aid to confirm a successful pairing.

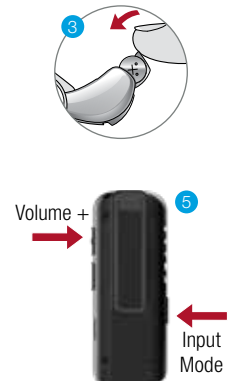


Pairing Remote Microphone + to Bluetooth Devices

- 1 Ensure the Remote Microphone + is powered on.
- 2 Press the Input Mode and Volume Down buttons simultaneously for 7 seconds until the LED status indicator rapidly blinks blue.
- 3 Put the Bluetooth device into pairing mode.
- 4 When the Remote Microphone + appears, select it on the Bluetooth device.
- 5 The Remote Microphone + LED will turn green for 2 seconds with successful pairing.

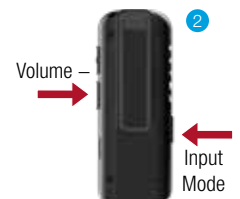

Use

- Press the Multifunction button for 2 seconds to power the Remote Microphone + on or off.
- The Remote Microphone + can be clipped to clothing or worn on a lanyard.
- Remote Microphone + operating range is up to 30 feet (10m). If the user walks out of range of the Remote Microphone + audio stream, streaming stops and will need to be manually restarted.
- The Remote Microphone + can stream for up to 10 hours on a full charge.
- The Remote Microphone + reaches a full charge in 3 hours.
- The LED status indicator will blink red and an audible indicator tone will play in the hearing aids every 5 minutes when the Remote Microphone + battery is low.
- The Remote Microphone + will momentarily mute if it is dropped and will resume streaming once the device is stabilized.
- The start/stop streaming function as well as the streamed audio volume level can be adjusted via the Remote Microphone +, hearing aid user controls, the 2.4GHz Remote Control or the Thrive Hearing Control App.
- When the audio source is set to Remote Mic or to Bluetooth for phone streaming, the Remote Microphone + should be within 8 inches (20cm) of the speaker's mouth. Wearing the Remote Microphone + in a vertical position provides the best performance (A).

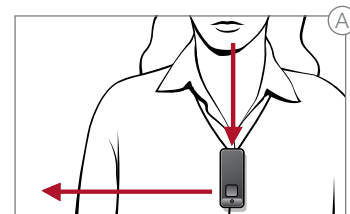

Input Mode


- 1 Press the Input Mode button to advance to the next available audio source. When cycling through input modes, the FM System and Line-In options will only be available when physically attached to the Remote Microphone +.
- 2 If the Input Mode LED is off, press the Input Mode button to temporarily illuminate the LED.
- 3 When the Remote Microphone + is powered off, it will remember the input mode that was last used and will return to that position when powered back on.



Audio Source Options


Remote Microphone:

Remote Microphone + automatically optimizes the microphone settings based on its physical orientation. Sound is received equally from all directions (omnidirectional) when used tabletop and prioritizes sound from above while reducing sound from other directions (directional) when worn on the body.

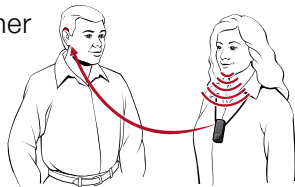

To use Remote Mic function:

- Ensure the Remote Microphone + is powered on and the input mode is set to Mic.
- Streaming starts automatically in the hearing aids.

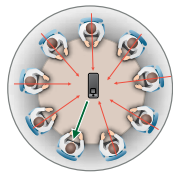

Incoming/Outgoing Phone Call Streaming:

- Ensure the Remote Microphone + is powered on and the status indicator LED is blue.
- The input mode does not matter as a phone call will interrupt other audio input modes.
- Press the multifunction button to accept the incoming call.
- For an outgoing call, dial the number on the mobile phone. Streaming starts automatically in the hearing aids.



- Press the multi-function button to end a phone call.
- Once the phone call has ended, the Remote Microphone + will automatically return to the previous input mode.
- To reject a phone call, press the multi-function button for more than two seconds.
- Phone calls will interrupt the Remote Microphone + audio stream. After the phone call concludes, all streaming stops. If desired, the Remote Microphone + audio stream will need to be manually restarted.

Other Audio Streaming

- Ensure the Remote Microphone + is powered on and the status indicator LED is blue.
- Set the input mode to Bluetooth.
- Start music or other audio on your Bluetooth device. Streaming starts automatically in the hearing aids.



Input Mode (Continued)

Audio Source Options

T Teleloop System

Remote Microphone + can receive audio from a Teleloop system.

Wearing the Remote Microphone + in a vertical orientation provides the best performance.




To stream Teleloop audio:

- Ensure the Remote Microphone + is powered on and the input mode is set to Teleloop.
- Streaming starts automatically in the hearing aids.

F FM System

- Insert the FM system transmitter into Europin Jack.
- Ensure the Remote Microphone + is powered on and the input mode is set to FM system.
- Streaming starts automatically in the hearing aids.

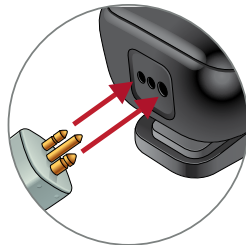


M Line-In

When a Line-In cable is attached via the microUSB Jack, the Remote Microphone + can receive audio from a non-Bluetooth-enabled mobile phone, MP3 player, tablet or computer.

To stream Line-In audio:

- Insert the Line-In cable into the microUSB Jack.
- Ensure the Remote Microphone + is powered on and the input mode is set to Line-In.
- Start music or other audio on your electronic audio source. Streaming starts automatically in the hearing aids.

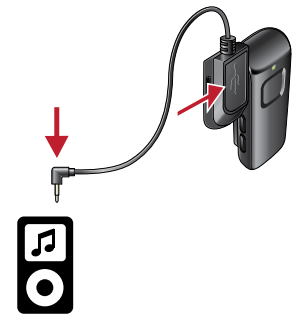

*Accessories only pair to hearing aid (not to iPhone/app).

Livio AI and Thrive are compatible with iPhone X, iPhone 8 Plus, iPhone 8, iPhone 7 Plus, iPhone 7, iPhone SE, iPhone 6s Plus, iPhone 6s, iPhone 6 Plus, iPhone 6, iPhone 5s, iPhone 5c, iPhone 5, iPad Pro, iPad Air 2, iPad Air, iPad (4th generation), iPad mini 4, iPad mini 3, iPad mini with Retina display, iPad mini, iPod touch (5th generation) and Apple Watch. Use of Apple Watch requires iPhone 5 or later.

Apple, the Apple logo, iPhone, iPod touch, Apple Watch, App Store and Siri are trademarks of Apple, Inc., registered in the U.S. and other countries.

Google Play and Android are trademarks of Google Inc.

© 2018 Starkey. All Rights Reserved. 8/18 FLYR3405-00-EE-ST