

SURFLINK MOBILE 2: AUDIO STREAMING WITH STREAM BOOST

PRODUCT AVAILABILITY: Z Series

SurfLink Mobile 2: Audio Streaming with Stream Boost assumes that SurfLink Mobile setup has been completed. Refer to the *SurfLink Mobile 2: Setup QuickTIP* available via Inspire Online for additional information.


StreamBoost is available in Z Series products to deliver an automatic and enhanced sound quality experience to the patient any time a streamed media input is detected.

STREAM BOOST

Stream Boost defaults ON when SurfLink Mobile 2 is enabled in **Inspire 2015.1**. If desired, disable Stream Boost through one of the following three methods:

NOTE: Stream Boost settings, such as the frequency response, noise control, and Multiflex Tinnitus, can be adjusted.


- » Select **Memories** from the left navigation bar, then select **Stream Boost: Off** from the Memory Environment options.
- » Select **Accessories** from the left navigation bar, then deselect the **Stream Boost** box under Configure Streaming Behavior.
- » Select the **Stream Boost memory tab** from any screen displaying the frequency response, then select **Stream Boost: Off**.

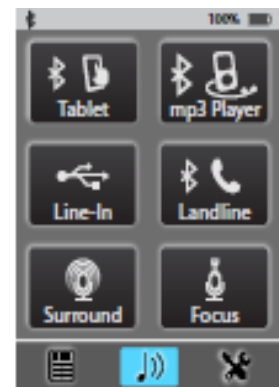

Streaming Screen

STREAMING AUDIO VIA BLUETOOTH

1. Power on SurfLink Mobile 2 by pressing the **Power** mechanical button.
2. Select the **musical note** icon at the bottom of SurfLink Mobile 2 to get to the **Streaming** screen.
3. Ensure the volume level from the source audio device is at **50%** or higher.
4. If wired input, attach input source to the SurfLink Mobile 2 by connecting the wired adaptor via the micro-USB port.
5. Select the icon for the desired audio device source to begin streaming. It will highlight **Blue** once active.
6. Use the **volume controls** on the side of the SurfLink Mobile 2 to increase or decrease the volume of the streaming input.
NOTE: The SurfLink Mobile 2 will remember the last volume setting for each source audio device.
7. Select **audio device source** icon to stop the streaming.

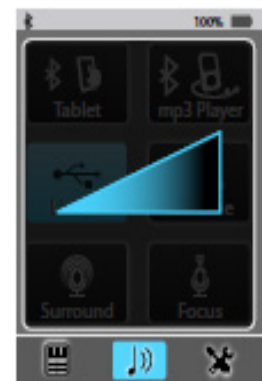

Streaming Volume Change

STREAMING AUDIO FROM SURFLINK MOBILE 2 REMOTE MICROPHONES

1. Power on SurfLink Mobile 2 by pressing the **Power** mechanical button.
2. Select the **musical note** icon at the bottom of SurfLink Mobile 2 to get to the **Streaming** screen.
3. Select the desired microphone:
 - » **Surround** (omni directional microphone) for quiet environments.
 - » **Focus** (directional microphone) for noisy environments.

NOTE: When Focus is selected, an instructional screen will momentarily appear explaining recommended orientation of the SurfLink Mobile 2.


4. Use the **volume controls** on the left side of the SurfLink Mobile 2 to increase or decrease the volume of the streaming input.

NOTE: The SurfLink Mobile 2 will remember the last volume setting for each source audio device.
5. Select **Surround** or **Focus** to stop the streaming audio session.

NOTE: Phone calls will take precedence over all streaming session types. Refer to the *SurfLink Mobile 2 Hands-Free Cell Phone Use QuickTIP* available via Inspire Online for additional information regarding phone use with the SurfLink Mobile 2.



Streaming Screen

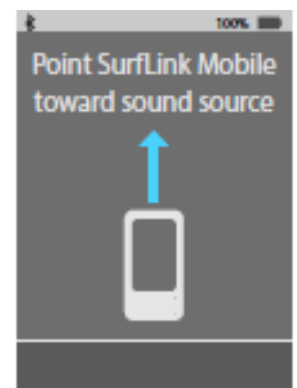

Focus Instructional Screen