

My Notes

Please provide the following information and **bring this brochure with you to your next appointment:**

My Apple device is:

I currently have the following Bluetooth® devices paired with my Apple device:

Halo 2 and TruLink are compatible with iPhone 7 Plus, iPhone 7, iPhone SE, iPhone 6s Plus, iPhone 6s, iPhone 6 Plus, iPhone 6, iPhone 5s, iPhone 5c, iPhone 5, iPad Pro, iPad Air 2, iPad Air, iPad (4th generation), iPad mini 4, iPad mini 3, iPad mini with Retina display, iPad mini, iPod touch (5th generation), and Apple Watch. Use of Apple Watch requires iPhone 5 or later. "Made for iPod," "Made for iPhone," and "Made for iPad" mean that an electronic accessory has been designed to connect specifically to iPod, iPhone, or iPad, respectively, and has been certified by the developer to meet Apple performance standards. Apple is not responsible for the operation of this device or its compliance with safety and regulatory standards. Please note that the use of this accessory with iPod, iPhone, iPad or Apple Watch may affect wireless performance. Apple, the Apple logo, iPhone, iPad, iPod touch, iTunes, Apple Watch and App Store are trademarks of Apple Inc., registered in the U.S. and other countries.



Starkey.com

TRULINK®

Preparing for your
Made for iPhone
fitting appointment



Congratulations!

This is an exciting time for you and your loved ones as you pursue better hearing. Your hearing care professional has placed your order for your Made for iPhone® hearing aids. Because of the unique way that these hearing aids connect with Apple® devices, there are some actions needed **before your next appointment.**



Please complete the following three steps:

STEP 1: Ensure that your Apple device has the latest version of iOS software.



1. To check if the latest version* of iOS software is installed on your Apple device:

Tap **Settings > General > Software Update.**

2. If a software update is available, tap **Install.**

NOTE: You must be connected to WiFi or your computer via iTunes to update iOS.

STEP 2: Download the TruLink Hearing Control app.

1. Tap the **App Store® icon.**
2. Type **TruLink** in the search field.
3. Once the TruLink Hearing Control app screen is displayed, tap **Get** (it will change to **Install**).



*Must be 9.0 or higher.

4. Tap **Install** to load the app on your device.

NOTE: Although this is a free app, you will be prompted to sign into your iTunes® account.

5. Type your Apple ID and password to complete the download process.

STEP 3: Ensure that your Apple device is compatible with your Made for iPhone hearing aids.

Visit TruLinkHearing.com/FAQ for a full listing of Apple devices that are compatible with Made for iPhone hearing aids.

If your current Apple device is not on this list, contact your hearing professional.

You have now completed all the steps necessary to prepare for your fitting appointment.