

Starkey Hearing Technologies Charger



The Starkey® Hearing Technologies Charger operates as a charging station, dry aid kit and case for all Starkey lithium-ion hearing aids. The Charger is equipped with LED lights to indicate charging status for both the hearing aids and the on-board battery.

LED Indicators for Hearing Aid Charging

The front of the Charger is equipped with two LED indicators to provide information about hearing aid charging.

- 1 When the hearing aids are inserted into the Charger, the front LED lights will glow green to indicate successful charging.
- 2 When the hearing aids are fully charged, the front LED lights will be solid green indicating a full charge.
- 3 If there is an issue with charging and the hearing aids are in fault state, the front LED lights will blink red.

NOTE: *If a fault state is detected, remove the hearing aids from the Charger, wait until the LEDs turn off, and re-insert the hearing aids into the Charger. If the fault state continues, contact your hearing professional.*



LED Indicators for On-board Battery

The back of the Charger is equipped with four LED indicators to provide information about the status of the on-board battery. The on-board battery allows charging of the hearing aids to occur while the Charger is away from a power source.

- Four solid green lights indicate the Charger is plugged into a power source and is fully charged.

NOTE: *When charging without the power cord, the back LEDs will turn off after 10 seconds to preserve battery life.*

- Four solid lights indicate the battery has more than 75% charge.
- Three solid lights indicate the battery has less than 75% charge.
- Two solid lights indicate the battery has less than 50% charge.
- One blinking light indicates that the charge is low, and the Charger should be connected to a power source to charge the on-board battery.

NOTE: *When plugged into a power source, both the hearing aids and on-board battery will charge simultaneously.*



Use

- If the Charger is not plugged in and runs out of charge, the hearing aids will turn ON in the charging case.
- It is safe to keep the hearing aids in the Charger as they cannot overcharge.
- Charging will occur with the lid open or closed.

NOTE: *It is recommended to keep the Charger lid closed to preserve the desiccant puck if also using as a dehumidifier.*