

# Starkey Hearing Technologies Mini Remote Microphone



## Pairing

Starkey® Hearing Technologies Mini Remote Microphone must be paired to interact with Livio® AI and Livio hearing aids. Pairing occurs once, then the hearing aids and Mini Remote Microphone will automatically recognize each other. The Mini Remote Microphone can be paired within Inspire® or outside of Inspire. Livio AI and Livio hearing aids can be paired with up to five 2.4 GHz Starkey Hearing Technologies streaming accessories, but only one can be actively streaming.

### Pairing within Inspire

- 1 Launch Inspire, connect and detect hearing aids.
- 2 Select “Accessories” on the left navigation bar.
- 3 Ensure the Mini Remote Microphone is powered on.
- 4 Place the hearing aids within 6 inches of the Mini Remote Microphone.
- 5 Place the Mini Remote Microphone in pairing mode by double clicking the power button. The LED status indicator will alternately blink blue and red.
- 6 Select “Pair” to put the hearing aids into pairing mode. (A)
- 7 Pairing is successful when the Mini Remote Microphone image displays on the accessory screen. (B)
- 8 Use the drop-down menu below the Streaming Accessories box to set “Hearing Aid Mic While Streaming.” Mic Reduced (50%) is the default. (C)
- 9 Select the Information icon for pairing instructions, if needed. (D)




#### Power On

Press the power button for 3 seconds and release when the LED illuminates.



#### Power Off


Press the power button for 2 seconds and release when the LED turns off.

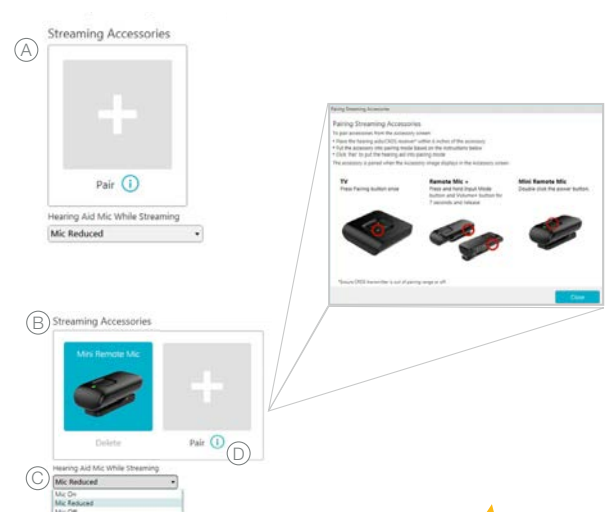


## Pairing (Continued)

### Pairing Outside of Inspire

- 1 Power off any mobile phone or other accessories that were previously paired with the hearing aids.
- 2 Ensure the Mini Remote Microphone is powered on.
- 3 Power off, then power on the hearing aids to place in pairing mode.
- 4 Place the hearing aids within 6 inches of the Mini Remote Microphone.
- 5 Place the Mini Remote Microphone in pairing mode by double clicking the power button. The LED status indicator will alternately blink blue and red.
- 6 A green LED will blink for each hearing aid to confirm a successful pairing.

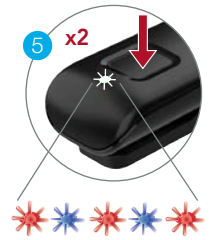


## How it Works

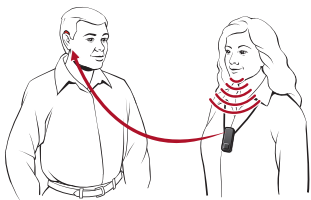

The Mini Remote Microphone allows the hearing aid wearer to focus on a single speaker when

background noise is present. While not optimized as a general purpose omnidirectional microphone for tabletop use, the Mini Remote Microphone may also provide benefit for television watching. Place the Mini Remote Microphone near the television speakers. To achieve a comfortable listening level, adjust the Mini Remote Microphone distance from the television speakers or modify the audio streaming volume level.

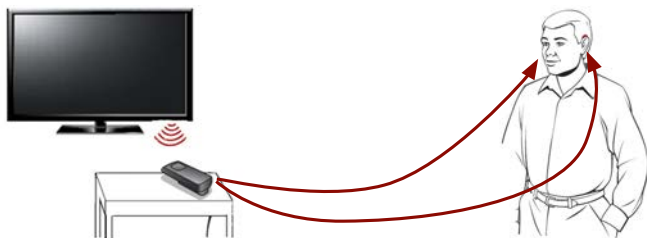

### To begin streaming from the Mini Remote Microphone:

- 1 Ensure the Mini Remote Microphone is powered on by pressing the power button for 3 seconds.
- 2 Streaming starts automatically in the hearing aids.
- 3 When finished, power off the Mini Remote Microphone to stop streaming.


### Audio streaming will stop when: \_\_\_\_\_

- The Mini Remote Microphone is powered off.
- A Phone call is received via a direct mobile phone stream into the hearing aids. After the phone call concludes, all streaming stops. If desired, the Mini Remote Microphone stream will need to be manually restarted.
- If the user walks out of range of the Mini Remote Microphone (up to 65 feet or 20 meters), streaming stops and will need to be manually restarted.

## Use

---

- The Mini Remote Microphone can be paired to one set of Livio AI/Livio hearing aids.
- The Mini Remote Microphone can be clipped to clothing or worn on a lanyard.
- The Mini Remote Microphone operating range is up to 65 feet (~20m).
- Place the Mini Remote Microphone close to the audio source for optimal results. The Mini Remote Microphone should be within 8 inches (20cm) of the speaker's mouth.
- The Mini Remote Microphone can stream for up to 10 hours on a full charge.
- The Mini Remote Microphone reaches a full charge in less than 3 hours and can reach a 50% charge in approximately 1 hour.

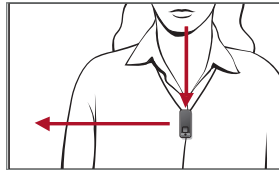

- When the Mini Remote Microphone battery is low, the LED status indicator will slowly pulse red and an audible indicator tone will play in the hearing aids every 5 minutes.
- The Mini Remote Microphone will momentarily mute if dropped and will resume streaming once the device is stabilized.
- Manually Start/Stop streaming via the hearing aid user controls, Starkey Hearing Technologies Remote Control, Thrive™ Hearing Control app and/or power the Mini Remote Microphone on or off.
- Stream Volume can be adjusted via the Thrive app, Starkey Hearing Technologies Remote Control and/or the hearing aid user controls.