

## SURFLINK MINI MOBILE

**PRODUCT AVAILABILITY:** Muse iQ, SoundLens Synergy iQ, Muse and SoundLens Synergy

The SurfLink Mini Mobile consists of the SurfLink Remote Microphone 2 and the SurfLink Mini Mobile Adapter.

The SurfLink Mini Mobile allows users to wirelessly stream phone calls and other audio from connected Bluetooth devices, such as cell phones, computers and MP3 players. The SurfLink Mini Mobile Adapter attaches to the SurfLink Remote Microphone 2.

The SurfLink Mini Mobile Adapter adds Bluetooth phone call and audio streaming capabilities to the SurfLink Remote Microphone 2.

### PRIOR TO USING THE SURFLINK MINI MOBILE

1. Select the check box next to SurfLink Mini Mobile on the Accessories screen within Inspire® to ensure customization of Stream Boost and hearing aid microphone offsets while streaming.
2. Ensure the hearing aids have been synced to the SurfLink Remote Microphone 2. Refer to the SurfLink Remote Microphone QuickTIP for additional information.

### SURFLINK MINI MOBILE: SETUP AND BASIC USE

1. Attach the SurfLink Mini Mobile Adapter to the female microUSB jack of the SurfLink Remote Microphone 2.

**NOTE:** The SurfLink Mini Mobile Adapter will only operate with the SurfLink Remote Microphone 2, which displays a white dot on the slider.

2. Ensure the SurfLink Mini Mobile Adapter and SurfLink Remote Microphone 2 are charged. The SurfLink Mini Mobile Adapter can be charged alone, or both the SurfLink Mini Mobile Adapter and SurfLink Remote Microphone 2 can be charged together while attached. Streaming can occur during charging.
3. To power SurfLink Mini Mobile Adapter ON, press its push-button for two seconds and release when the green LED appears. To power SurfLink Mini Mobile Adapter OFF, press the push-button for two seconds and release. The red LED appears briefly to provide confirmation.
4. The SurfLink Mini Mobile Adapter may be paired with up to two Bluetooth devices. Audio will only stream from one Bluetooth device at a time.




Connect SurfLink Remote Microphone 2 and SurfLink Mini Mobile



Power On (green)

## PAIRING SURFLINK MINI MOBILE

1. Place the Bluetooth device into pairing mode. Ensure the SurfLink Mini Mobile is powered ON and not Bluetooth-connected to other devices (the green LED will be slowly pulsing when on).
2. Double-click the SurfLink Mini Mobile Adapter push-button. The LED will begin to rapidly blink blue, which places the SurfLink Mini Mobile Adapter into Bluetooth pairing mode.
3. On the Bluetooth device, select the SurfLink Mini name to complete the pairing process. The SurfLink Mini Mobile Adapter LED will display green for two seconds, confirming a successful pairing. Pairing between SurfLink Mini Mobile Adapter and the Bluetooth device only needs to occur once. They should automatically Bluetooth connect when both are powered ON and within wireless operating range. Turning Bluetooth OFF from the paired device will break the streaming connection.

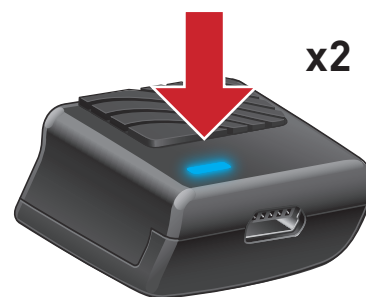

Pairing Mode

## STREAMING PHONE CALLS

Ensure the SurfLink Remote Microphone 2 and SurfLink Mini Mobile Adapter are attached and both powered ON.

1. Accept an incoming phone call: When the phone ringtone is heard through the hearing aids, briefly Press and Release the push-button on the SurfLink Mini Mobile Adapter. During the call, the SurfLink Mini Mobile Adapter will pick up the user's voice and transmit it back through the phone to the other person on the call.

**NOTE:** For a no-look, hands-free conversation, the user can wear the SurfLink Mini Mobile System on a lanyard around the neck, clipped to their lapel, or placed near their mouth.

2. Place an outgoing phone call from a connected Bluetooth phone: Initiate a call from the connected Bluetooth phone. The call will automatically stream to the hearing aids.
3. Reject an incoming call from a connected Bluetooth phone: When the phone ringtone is heard through the hearing aids, press the push-button on the SurfLink Mini Mobile Adapter for more than two seconds. Release the button after the ringtone stops.
4. Hang up an active phone call: During the active phone call, briefly Press and Release the push-button on the SurfLink Mini Mobile Adapter.
5. Transfer a phone call: During an active phone call, Press the push-button on the SurfLink Mini Mobile Adapter for more than two seconds to move the call from the SurfLink Mini Mobile Adapter to the phone. This allows the call to be directly conducted from the phone, if desired. Re-press the push-button for more than two seconds to transfer the call back to the SurfLink Mini Mobile Adapter.

**NOTE:** When the SurfLink Remote Microphone 2 is ON and the SurfLink Mini Mobile Adapter is OFF, the system will function as a remote microphone and cannot stream audio from paired Bluetooth devices.


## STREAMING MUSIC AND OTHER AUDIO

Ensure that the SurfLink Remote Microphone 2 and SurfLink Mini Mobile Adapter are attached and both are powered ON.

1. Start playing music or other audio from the connected Bluetooth device. Audio streaming will automatically transmit.
2. Press Stop on the media player of the Bluetooth-connected device to halt the stream.

**NOTE:** While streaming phone calls or other audio, the hearing aid microphones will follow the settings programmed on the Accessories screen within Inspire.

To adjust the streamed phone call or other audio volume: adjust the volume on the Bluetooth device or adjust the volume on the hearing aids. This requires the volume control to be enabled for the hearing aids during Inspire programming.

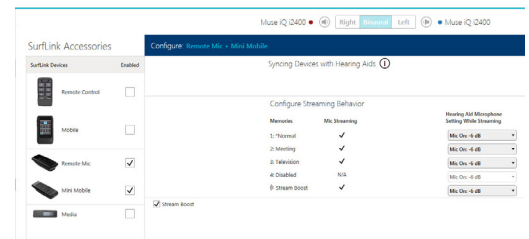

Hearing Aid Microphone settings while streaming