


Halo Programming Made Easy

Product Availability: Halo RIC 13 i110, i90, i70

Halo can be programmed with either non-wireless or wireless programmers. The patient's Apple device should be powered down prior to programming Halo. Programming will need to be completed prior to connecting Halo to the Apple Device. For further information see [Halo Set-Up Made Easy](#) QuickTIP.

Wireless Programming

1. Connect TruLink programmer to fitting computer via USB cable from docking station (See Fig. 1)
2. Follow all on-screen instructions to install the programmer
3. Select the patient within Inspire
4. Click on **New Session** on the bottom right of the Devices Detected box on the Get Started Screen.
5. Click **Start**

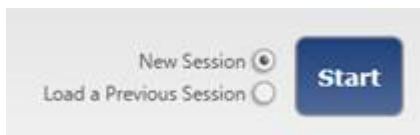

6. **Auto Path** will launch, ensure all icons are **checked** and highlighted in **orange**
7. Click **Begin** to start the routine
8. Follow all onscreen prompts
9. Make sure **Receiver Matrix** and **Acoustic Options** selection in Inspire match device configuration
10. Choose the **Voice Language** for Indicators
11. Click **Next**
12. Select **Experience Level**
13. Click **Close** to exit Auto Path and open Quick Fit

Non-Wireless Programming

1. Connect Halo devices to the SpeedPort, NOAHLINK or HiPro using the programming cable and flex strips, as shown (Fig. 2)
- Note:** Keep the battery door **OPEN** when programming
2. Follow steps 3-13 above to complete programming

- The TruLink Hearing Control App may be downloaded from the Apple App store.
- Applicable Apple devices for use include:
iPhone 4s, 5, 5s or 5c
iPad 4 or newer, iPad mini, iPad Air
iPod Touch 5th generation or newer
- Apple iOS 7.1 or higher is required.
- A fresh battery is necessary to program Halo wirelessly
- Halo devices will auto-populate within the Inspire Get Started screen
- Programming will need to be completed prior to connecting the Halo devices to the Apple device
- Expected programming range will be about 10m or 30ft.



Figure 1



Figure 2

Important Follow-up Programming Requirements

- Turn **OFF** the Apple Device Halo is paired to
- The Device Detected screen will not appear unless this action occurs

Good-To-Know

- Halo hearing aids may be paired with up to 5 Apple devices at a time. If additional devices are paired, original devices will be removed in sequential order. This means that the first device paired will be the first device removed if the 5 device pairing limit is exceeded.
- Halo can only be connected to one device for streaming at a time. To transition from one device to another, turn Bluetooth OFF on all paired devices within range that you do not wish to connect to