


Halo Set Up Made Easy

Product Availability: Halo RIC 13 i110, i90, i70

Pairing should be done **after** the hearing aids have been programmed in the Inspire Software, please see the ***Programming Halo*** QuickTIP for instructions on programming.

Pairing Halo to an Apple Device

1. Navigate to the **Settings** icon on the Apple device 
2. Select **General > Accessibility > Hearing Aids**
3. If Bluetooth is turned **Off** on this screen, turn it **On**
4. Open and close the battery door on the hearing aids

Note: A fresh battery is needed in each hearing aid

5. Tap on the **Hearing Aid Name** to connect the hearing aids to the Apple device
6. Select **Pair** for each request, if pairing a binaural set you will receive two requests.


Note: These requests may be several seconds apart

7. A black check mark will appear next to connected hearing aid name

Good-To-Know

- Halo can be paired with up to 5 Apple devices but can only be connected to one Apple device at a time
- To switch to another paired Apple device, turn Bluetooth off on the original device and the next Apple device will automatically be seen, granted Bluetooth is ON

- The TruLink Hearing Control App may be downloaded from the Apple App store.
- Applicable Apple devices for use include:
iPhone 4s, 5, 5s or 5c
iPad 4 or newer, iPad mini, iPad Air
iPod Touch 5th generation or newer
- Apple iOS 7.1 or higher is required.

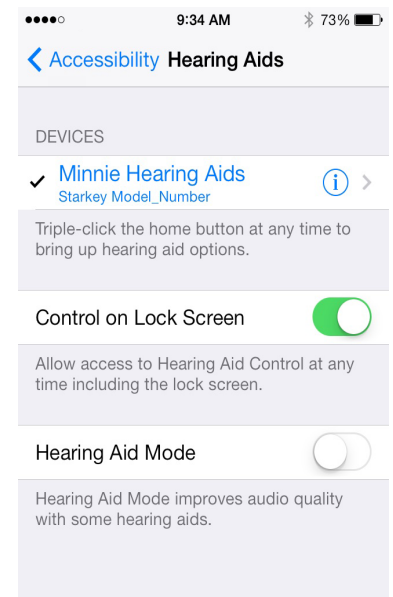

Halo successfully connected to an Apple device