

QuickTIPS



MUSE AND SOUNDLENS SYNERGY INITIAL FITTING PROTOCOL

PRODUCT AVAILABILITY: Muse and SoundLens Synergy Devices

GET STARTED

1. Launch **Inspire X**.
2. On the **Get Started** screen, select **View Programmers**, then choose the appropriate programmer and select **OK**.
3. Connect hearing devices to the fitting computer via the appropriate programming accessories. Refer to the **Cables and Connectors Guide** located under **Help** in Inspire for additional information regarding connecting the hearing devices.
NOTE: If you are fitting a hearing device on the right ear, select **Assign Right** to designate a hearing device for the right ear.
4. Select the **check box** next to the hearing device(s) to be programmed.
5. Select **Read Device Settings** to begin a new fitting or for a follow-up fitting session using the current settings in the hearing devices, or **Load a Previous Session** to retrieve a previously saved session, then select **Start**.

A product line featuring extended bandwidth, additional channels and bands, ear to ear phone streaming, a dedicated music memory, advanced classification to manage all environments, tinnitus stimulus in all devices, CROS/BiCROS devices, and wireless connectivity to SurfLink accessories.

INITIAL FIT AND AUTO PATH

Initial Fit launches automatically when programming new hearing devices.


1. Select the appropriate **Acoustic Options**, then select **OK**.
2. Select the appropriate **Experience with Amplification**. Refer to the **Experience Manager QuickTIP** for additional information.
3. Insert hearing devices into the patient's ears, reduce all sources of noise, then select **Begin** to run the Auto Path.
NOTE: If hearing device programming is conducted prior to the fitting appointment, re-run the Feedback Canceller with the hearing devices in the patient's ears.
4. Review the Auto Path results screen. Select **Restart** if Status is Incomplete for any steps, or select **Close** to continue.

USER CONTROLS


The User Control Configuration demo box will appear upon initial programming with Inspire X. Select **Show Demo** for demonstration of user control configuration options, or select **Don't show again** and **Close**. Refer to the **User Controls QuickTIP** for additional information.

QUICKFIT AND FINE TUNING

- » Select **QuickFit** on the left navigation bar to make global fitting adjustments. Refer to the **QuickFit and Fine Tuning QuickTIP** for additional information.
- » Select **Fine Tuning** on the left navigation bar or from the bottom of the QuickFit screen to make detailed fitting adjustments of individual channels and input levels. Refer to the **QuickFit and Fine Tuning QuickTIP** for additional information.

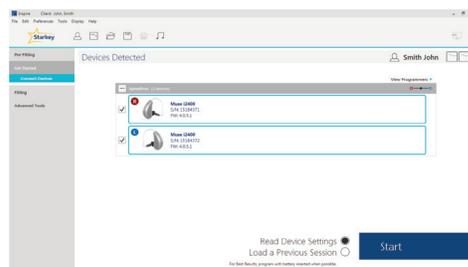

Get Started

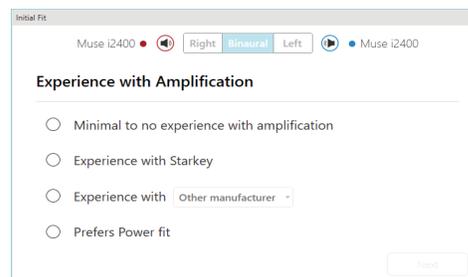


Initial Fit



Fine Tuning

MEMORIES

1. Select **Memories** on left navigation bar or select **Memories** from the middle of all fitting screens.
2. Select **drop-down arrows** to extend the Memories menu, then select the desired memories to program.
3. If configuring a **Music Memory**, refer to the *Music Memory QuickTIP*.
4. If enabling a **Telephone, Autophone, Telecoil, Autocoil, or Loop Memory**, select **Configure Phone and Loop Options** from the Memories screen. Refer to the *Phone Solutions QuickTIP* for additional information.
NOTE: Select **microphone offset** if enabling a Loop or Phone memory.
5. Select **Additional Configuration Options** from the Memories screen to Copy, Move, or Link memories and adjust the Telephone Release.

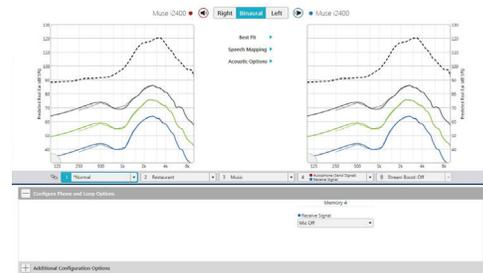

Memories

ACCESSORIES

Select **Accessories** on the left navigation bar to configure SurfLink accessories. Refer to the *SurfLink QuickTIPs* for additional information.

INDICATORS

1. Select **Indicators** on the left navigation bar.
2. Select or de-select **check boxes** to enable or disable indicators.
NOTE: Volume indicators may be enabled or disabled on an individual basis.
3. Select **Adjust All Indicators** to increase or decrease volume level for all Tone or Speech/Chimes indicators in 5 dB increments, or to change the voice language.
4. Select the **Individual Indicator names** to adjust the volume level and stimulus type for individual indicators, then select **Play** to play indicator directly through devices or **Demo** to play indicator through loudspeaker for demonstration purposes.



Indicators

ADDITIONAL ADJUSTMENTS

Refer to the *QuickTIP* library for additional information regarding programming of Experience Manager, Frequency Lowering, Environment Manager, and Tinnitus.

FITTING SUMMARY

1. Select **Fitting Summary** on the left navigation bar.
2. Select **feature names** or **details** to change status of features if necessary. Feature names are hyperlinks and will navigate to the adjustment screens.
3. Select **Print** button to generate a one page **Fitting Report** or a personalized **Device Guide**.
4. Select **Save** button to save current fitting session.



Fitting Summary

READY. SET. HEAR.

Ready. Set. Hear. is a one year triggered email program created to help new patients successfully transition into their new hearing devices. Contact your Marketing Representative for additional information.