

Downloading the TruLink Hearing Control App

Product Availability: Halo RIC 13 i110, i90, i70

Starkey has developed a free App that will turn an Apple Device into a remote control, personal adjustment tool and a remote microphone to give patients the ability to personalize their listening experience. The TruLink Hearing Control App may be downloaded via a computer and synced to an Apple Device or downloaded directly to the device itself.




- The TruLink Hearing Control App may be downloaded from the Apple App store.
- Applicable Apple devices for use include:
 - iPhone 4s, 5, 5s or 5c
 - iPad 4 or newer, iPad mini, iPad Air
 - iPod Touch 5th generation or newer
- Apple iOS 7.1 or higher is required.

Required Software

1. You will be required to have an iTunes account to download the TruLink Hearing Control App. If you do not have an iTunes account follow steps 2 and 3. If you already have an existing iTunes account, proceed to the Downloading the TruLink Hearing Control App section.
2. To set up an iTunes account, access iTunes at: www.itunes.com
3. Select the iTunes **Download** button to begin the download and installation process



Downloading the TruLink Hearing Control App

1. Tap the App Store icon 
2. Type TruLink in the Search Field
3. Tap **Free** 
4. The free button will then convert to **Install**, tap **Install** 
5. Sign In to iTunes Store
6. Enter your Apple ID password to complete the download process
7. The TruLink Hearing Control App is now ready to use on your Apple Device



NOTE: For a complete guide to the TruLink Hearing Control App, refer to *Halo Set Up Made Easy* QuickTIP

