

TruLink Hearing Control Phone Calls Made Easy

Product Availability: Halo RIC 13 i110, i90, i70

The TruLink Hearing Control App in concert with Halo hearing aids enables hands-free phone conversation with an iPhone.

NOTE: Before the TruLink Hearing Control App may be used, it must be downloaded from the Apple App Store and the hearing aids must be professionally programmed and paired with an Apple Device. See the *Halo Programming Made Easy* QuickTip for additional information

- The TruLink App may be downloaded from the Apple App store.
- Applicable Apple devices for use include:
 iPhone 4s, 5, 5s or 5c
 iPad 4 or newer, iPad mini, iPad Air
 iPod Touch 5th generation or newer
- Apple iOS 7.1 or higher is required.
- Expected streaming range will be about 10m or 30ft.

Placing an Outgoing Call with Halo

1. Tap the Phone icon.
2. Tap on the appropriate contact to initiate the call.
3. Once the phone begins to ring, the Source display appears, indicating the Halo hearing aids are active. (Fig.1)
4. The Source of the call may be changed from the hearing aids to the iPhone or Speaker function.
5. Tap to disconnect the call.

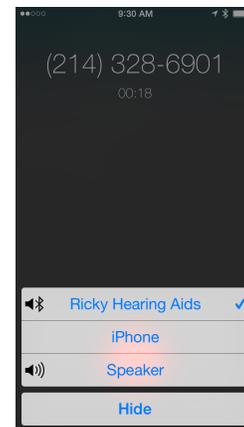


Figure 1

Accepting an Incoming Call with Halo

1. iPhone will indicate the arrival of an incoming call. (Fig.2)
 If the call is received while streaming media, the stream will be interrupted by the call.
2. Once the call is accepted, the iPhone screen will display the Call Options dialogue box. (Fig. 3) Another option may be chosen during the course of the phone call.
3. Tap to disconnect the call.



Figure 2

Figure 3

Using Halo with iPhone 4s

To enhance the streaming capability with an iPhone 4s, the iPhone setting should be set to mono audio.

1. Tap the Settings icon>General>Accessibility> Hearing Aids>Mono Audio. Slide right to enable. (Fig. 4)
2. This will send both the left and right signals to the chosen ear.
3. The Halo wearer can designate which ear from the Native Controls screen by enabling the slider to stream to the right or left hearing aid. (Fig 5.)

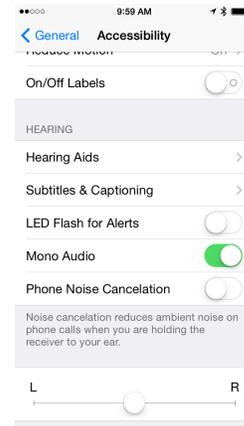


Figure 4

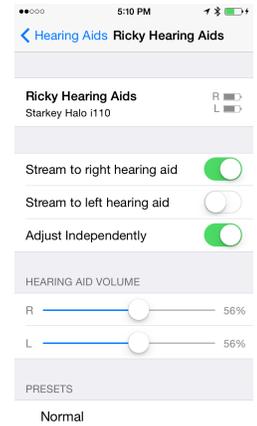


Figure 5

- Phone calls will take precedence over all streaming sessions.
- If an incoming call is received while streaming, the streamed audio is paused while the call is taken. Once the call has been disconnected, the music will resume at the point at which it stopped to handle the phone call.
- If the iPhone is carried on the body, response is optimized with the back of the iPhone facing away from the body.
- If the iPhone will be placed in a pocket, a higher pocket is recommended to provide closer proximity to both ears for binaural streaming.